Xailin Wash

Sterile, preservative free solution for ocular cleansing in unit dose vials. Not for injection

EN

COMPOSITION

Sodium chloride, Boric acid, Borax, Rose water, purified water qsd 5 ml.

DESCRIPTION OF THE PRODUCT

Xailin Wash is a sterile solution for ocular cleansing in unit dose vials of 5 ml.

INDICATIONS

Xailin Wash is recommended for:

- Treatment of superficial irritations of eyelids and eyeball: ocular cleansing solution facilitates elimination of foreign bodies which irritate the eye, such as sand, mites, dusts, pollens, fragments of make-up and eases the sensation of discomfort.

- Adjuvant treatment for infection, allergy or inflammatory pathology of eyelids, conjunctiva and cornea.
- Daily and post-surgical eye cleansing (children and adults).
- Xailin Wash assists in eye hydration.

PRECAUTIONS AND WARNINGS

Do not inject, do not swallow.

Keep out of the reach of children.

If you are a contact lens wearer, you can use this product but you will need to remove your contact lenses beforehand. Single use only: any solution not used immediately after opening should be discarded.

One unit dose vial can be used for treatment of both eyes.

Wash your hands thoroughly before using the solution.

Do not touch eyelid surface with the tip of container.

The efficacy of an eye drop or an eye ointment can be reduced by simultaneous instillation of the eyewash. In case of concomitant treatment, it is advisable to wait 10 minutes before each instillation and to use Xailin Wash first. In case of occurrence, persistence or worsening of an irritation or an excessive tearing or eye disorders, stop the use of the solution and consult your practitioner.

STORAGE

Do not use after the expiry date shown on the carton or printed on the label.

Discard the vial after single use.

Do not reuse a single use container opened.

As this medical device does not contain any preservative, a re-use would lead to the risk of administration of a contaminated solution.

Before opening, store at room temperature.

HOW TO USE XAILIN WASH?

2 up to 8 eye washes daily.

Make sure before use that the single unit is not opened or damaged. Tear off one single-use container from the strip, twist top to open.

The eyewash is applied directly by pressing slightly on the vial. It is recommended to pull the lower eyelid downward, glance to the top, then pull the upper eyelid upward and glance below to spread the solution onto the entire eye surface. This will eliminate the foreign bodies which irritate the eye, increase the volume of the lachrymal film and hydrate tissues.

Xailin Wash is a class IS medical device.

CE mark certification was obtained on 18/04/2006. Date of revision of the patient instruction leaflet: November 2012.

XAILIN is a trademark of Nicox

 Distributed by **Nicox Pharma**, CS 10313, Sophia Antipolis, 06560 Valbonne, France - nicoxpharma@nicox.com

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