NEOVIS®

LUBRIFIANT OCULAIRE

FR

PRESENTATION :

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

ACL : 3401060025687

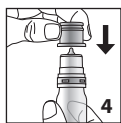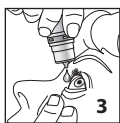CE0459

3. Instill 1 or 2 drops to the eye.
4. Replace the protective cap on the dispenser after every use.

PRECAUTIONS FOR USE

Store between 2 and 25 °C.

To avoid all risk of product performance loss and microbial contamination:

- Do not touch the tip of the dispenser nozzle.
- Do not touch the eyes and eyelids with the tip of the dispenser nozzle.
- Replace the protective cap on the dispenser nozzle after every use.
- Do not use if the dispenser is damaged.
- Do not use if the tamper-proof ring is absent or broken.
- Use within 3 months after opening the dispenser for the first time.
- Do not use if the expiry date indicated on the packaging has been passed.

If your doctor has prescribed eyewash, allow a minimum period of 15 minutes between application of the two eyewashes (there is a risk of modifying the effect produced by the eyewashes).

If problems persist while using NEOVIS®, consult a doctor.

Keep out of the reach and sight of children.

CONTRA-INDICATIONS

Do not use in cases where there is hypersensitivity to one of the components.

ADVERSE EFFECTS

In very rare cases, transitory problems such as mild conjunctival irritation, the feeling of a foreign body in the eye, redness or a burning sensation, or momentary disturbance of vision, can appear on application.

If an adverse effect persists, or if there is an adverse effect not described in this leaflet, stop using the product and consult your doctor.

Medical device compliant with the requirements of the amended directive 93/42/EEC.

NEOVIS® is a class IIb medical device.

Date of last revision of package insert: 2015.

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EN

INTRODUCTION :

NEOVIS® is a sterile solution of 0.18% sodium hyaluronate, presented in a 10 ml multidose dispenser. It does not contain a preservative.

COMPOSITION :

0.18% sodium hyaluronate, sodium chloride, potassium chloride, disodium phosphate, sodium citrate, magnesium chloride, calcium chloride, purified water.

IN WHICH CIRCUMSTANCES SHOULD NEOVIS® BE USED?

NEOVIS® is recommended in cases where there are moderate or severe feelings of ocular dryness.

NEOVIS® can be used by contact lens wearers (rigid or soft).

NEOVIS® contains sodium hyaluronate, a component that is naturally present in the eye, which helps to protect the ocular surface in case of irritation. The physical properties of sodium hyaluronate give NEOVIS® a capacity for water retention, which allows a lubricating film to be formed on the eye surface.

NEOVIS® is hypotonic. It does not contain a preservative agent.

HOW SHOULD NEOVIS® BE USED?

NEOVIS® is administered via the ocular route.

Wash hands carefully before handling the dispenser.

When using for the first time, check that the tamper-proof ring is intact, then break the seal.

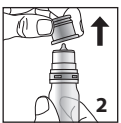
Remove the protective cap.

With the index finger, draw the lower eyelid gently downwards, while looking up, then squeeze the dispenser (positioning fingers in its centre) to apply the eye drops.

Unless advised to the contrary, apply 1 or 2 drops as often as necessary.

After application, a few natural blinks of the eyelids will distribute the solution over the eye surface.

Replace the protective cap after every use.

1. When using for the first time, check that the tamper-proof ring is intact, then break the seal.
2. Remove the protective cap.

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