DE

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EN

How Gynofit Moisturising Gel works

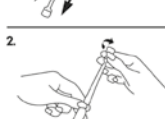
Gynofit Moisturising Gel is an aqueous vaginal gel that forms a thin film of moisture on the vaginal wall and thus moisturises the mucous membrane.

Intended use

Gynofit Moisturising Gel is used for symptomatic alleviation of vaginal dryness (especially if the mucous membrane is thin, susceptible to attack and vulnerable) and the associated discomfort. Ideal after childbirth, during menopause or cancer therapy. Furthermore, symptoms that are regarded as irritative, such as burning or itching, are alleviated, thus enhancing physical well-being. Gynofit Moisturising Gel is also ideal for facilitating sexual intercourse. Gynofit Moisturising Gel guards against irritation.

Method of use and dose

Each tube contains one application dose. The gel is directly introduced to the vagina from the specially shaped, sealed applicator.

	1. Shake the tube contents towards the tip of the tube
	2. Break off the tip of the tube by twisting it off
	3. Introduce the applicator to the vagina
	4. Squeeze out the contents into the vagina

Side effects

When Gynofit Moisturising Gel is used correctly according to the manufacturer's recommendations, its composition is such that no side effects are to be expected beyond an initial but reversible burning sensation and/or itching. In predisposed individuals, especially if the vaginal mucosa is already damaged owing to unsuitable care, hygiene or local contraceptives for example, mucosal irritation or allergic reactions can also occur in isolated cases.

Contraindications

Do not use Gynofit Moisturising Gel if you are known to be allergic to any ingredients; if such reactions are triggered, immediately discontinue using the gel; consult a doctor if necessary.

Use

Depending on needs and the amount of dryness (which is quite different in every woman), use an applicator on a daily basis, ideally during the day.

Notes

Gynofit Moisturising Vaginal Gel can be used for up to 30 consecutive days. There should then be a break of several days before recommencing use.

Gynofit cannot be used for contraception and it does not contain any spermicidal ingredients.

Gynofit Moisturising Gel can be used together with condoms.

Store out of children's reach.

Empty the tubes completely and do not use opened tubes a second time. Do not use damaged applicators.

Store at room temperature (15 - 25 °C). For the use-by date see packaging.

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Tentan AG, Dellenbodenweg 8, CH-4452 Itingen
Tentan Deutschland GmbH, Schloss Beuggen 3, 79618 Rheinfelden, Deutschland