# Holthaus Medical

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Aqua, Alcohol, Propylenglycol, Glycerol, Polyethylenglycol-monodecylether, Formic acid, PVP, 2-isopropyl-5-methylphenol, viscose

## D Wundreinigungssset

### Produktbeschreibung

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## GB Wound Cleansing Set

### Product description

Contaminations, which enter the wound, are the main reason of wound healing problems. Mechanical removal of these contaminations immediately after injury is therefore a suitable means to promote wound healing.

### Application

1. Remove the wipe from the packaging and unfold it.

2. Spray the fluid onto the wipe until it is well moistened.

*Note:* Preferably use too much fluid to achieve a better removal of contaminants in the wound.

3. Carefully clean the wound superficially (for example, by dabbing or wiping away from the wound).

*Note:* Do not spray the fluid directly onto the wound.

### Notes

Use the product only for the initial treatment of superficial wounds. If the depth of the wound is cannot be determined, the wound is heavily contaminated, or bleeding profusely, consult a physician.

Do not apply the fluid directly to the wound. Use the fluid only in combination with the enclosed wipe. Check your tetanus vaccination status, as there is a risk of tetanus infection from any wound. If in doubt, contact your physician.

If the wound does not heal and, for instance, shows signs of inflammation (e. g. redness), contact your physician – the product is not suitable for regular wound care during the healing process.

The product is suitable for single use only. Properly dispose of the wipe after use and do not reuse it. If the packaging of the product is damaged, do not use the product.

Current experiences with pregnant and nursing women using this product do not indicate a hazard. Accumulation of the fluid in breast milk is unlikely, as there is no distribution in the body due to the short contact time with the wound cleansing wipe.

### Contraindications

Do not use on the eye. As a precaution, do not use on babies and infants under three years of age, likewise not on persons who have a proven allergy (contact sensitization) against one of the ingredients specified above.

### Storage

Keep product out of reach of children. Do not use the product after the expiration date.