**ELEKTROLYT-SALBE S**

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# ELEKTROLYT-SALBE S

Medical Device

**NAWA® MEDICAL ELEKTROLYT-SALBE S** is a wound healing cream for external use for the healing of injuries to the skin.

- Promotes wound healing
- Reduces swelling
- Alleviates pain
- Reduces scar formation
- Infection prevention
- Suitable also for children

**Indications:** Injuries of the skin (e.g. cuts, abrasions, lacerations and contusions), 1st and 2nd degree burns (also for sunburns), irritation or inflammation of the skin at the anus (e.g. anal fissures, hemorrhoids stadium 1), for the treatment of scar tissue and blunt traumas. Also, for the care of exposed or dry, cracked and aged skin.

**Characteristics:** **NAWA® MEDICAL ELEKTROLYT-SALBE S** creates an optimal physical environment for the healing of injuries to the skin. Thus hematoma and edema were degraded, healing accelerated, swelling reduced and thereby pain relieved. The formation of new tissue is supported, the healing after surgeries or injuries (wounds, burns, sunburns) enhanced and scar formation reduced.

**Directions for use:** Wash hands before and after treating a wound. If necessary clean the wound carefully (e.g. with **NAWA® Wundspray**). Apply **NAWA® MEDICAL ELEKTROLYT-SALBE S** generously onto wound area and over the wound margin. Repeat several times a day for the first three days after injury.

**NAWA® MEDICAL ELEKTROLYT-SALBE S** may be used in combination with a customary dressing. By covering injured skin with a bandage the wound is protected additionally from external influences which could disturb the healing process.

**NAWA® MEDICAL ELEKTROLYT-SALBE S** contains no skin irritating substances and can be used on the mouth and on mucous membranes (e.g. nose).

**Duration of use:** The duration of use is based upon type and severity of the injury.

**Contraindication and side effects:** Do not use with hypersensitivity to any of the ingredients. A doctor should be consulted with burns second b or third degrees. Application for pregnant or breastfeeding women, babies and toddlers/young children should only be carried out with mandatory indication after consultation of a doctor.

**General safety instructions:** For external use. Do not swallow. Look after hygienic extraction of the salve. Protect against frost and heat. Close tube after use. Do not use damaged packages. Keep out of the reach of children. Protect from sunlight. Do not use after expiration date. Consider temperature limit. Keep in a dry place. Consider instructions for use. Pay attention to intact sealing.

**Shelflife:** 3 years after manufacturing

**package sizes:** 100 g, 20 g tube; 5 g sachet


**Time limit to use: tube:** Once opened **NAWA® MEDICAL ELEKTROLYT-SALBE S** should be used within 3 months.  
**/ Sachet 5g:** The product **NAWA® MEDICAL ELEKTROLYT-SALBE S Sachet** is intended for single use only. Once opened **NAWA® MEDICAL ELEKTROLYT-SALBE S Sachet** should be used immediately to avoid contamination.

**Batch code/  and Expiry date/ ** : printed onto bottom of folding box and stamped onto tube fold/sachet edge

**Inhaltsstoffe/Ingredients (INCI):**

Aqua purificata, Octyldodecanol, Hydrogenated Palm Glycerides, PEG-200 Hydrogenated Glyceryl Palmate, Cetyl Alcohol, Methylparaben (and) Propylparaben (and) Sorbic Acid (and) Dehydroacetic Acid, Trace elements (zinc, iron)

CE 0197

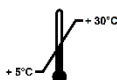e 100 g

e 20 g

e 5 g Sachet

**NAWA® HEILMITTEL GmbH**  
 Ostendstraße 100 · D-90482 Nürnberg  
 Telefon: +49 (0) 911-544 41-0  
 Telefax: +49 (0) 911-544 41-11  
 E-Mail: info@nawa.com  
 www.elektrolyt-salbe.de

Made in Germany

REV 052020