# Prontosan® Wound Spray

Read instruction for use  
Vor Anwendung Gebrauchsinformation lesen  
Lea las instrucciones de uso  
Attention, voir la notice d'instructions

Unique Device Identifier  
Einmalige Produktkennung  
Identificación única del producto  
Identifiant unique du dispositif

Medical Device  
Medizinprodukt  
Dispositivo médico  
Dispositif médical

Catalog number  
Katalognummer  
Número de referencia  
Référence d'article

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## Instructions for use

### en Prontosan® Wound Spray

**Intended purpose:** For cleansing, moistening and decontamination of:  
a) superficial acute and superficial chronic wounds (e.g. skin lesions, lacerations, abrasions, crush wounds).  
b) superficial burns.

For the cleansing of entry ports such as transurethral or suprapubic urological catheters, PEG-/PEJ-tubes and peristomal skin areas.

**1. Introduction:** Wounds require proper cleansing as they are generally contaminated with debris and microorganisms. These contaminants can interfere with the normal wound healing process and lead to complications such as infection. Prontosan® Wound Spray is able to remove these barriers to wound healing through its cleansing activity. Because it does not contain perfumes or dyes, it leaves no marks on the skin or tissues. Prontosan® Wound Spray is dermatologically tested. Due to the unique combination of ingredients (i.e. the antimicrobial substance, polihexanide, and the surfactant, betaine), Prontosan® Wound Spray is ideal for the prevention of biofilm formation. The summary of safety and clinical performance (SSCP) is available in the European database on medical devices (Eudamed) and can be found using basic-UDI-DI: 4039239000009252U or the product name. <https://ec.europa.eu/tools/eudamed>. Or requested from the Manufacturer.

#### Characteristics:

- Supports fast healing by optimal wound cleansing.
- Reduces odour.
- Prevents wound infection.
- Prevents biofilm formation.
- Reduces pain.
- Painlessly applicable.
- Non-irritant and well-tolerated.
- No inhibition of granulation and epithelialisation.
- Evaluated as dermatologically uncritical.
- Good compatibility with catheter and probe materials.

#### 2. Product profile and areas for use:

For cleansing, moistening and decontamination of:

a) superficial acute and superficial chronic wounds (e.g. skin lesions, lacerations, abrasions, crush wounds).  
b) superficial burns.

For the cleansing of entry ports such as transurethral or suprapubic urological catheters, PEG-/PEJ-tubes and peristomal skin areas. Moistens wound dressings and dissolves encrusted bandages or wound dressings during dressing changes.

**3. General use:** Extensively spray Prontosan® Wound Spray onto the entire wound and the wound site. With this, the risk of infection is reduced, and optimal healing conditions are generated. Afterwards, cover the wound with a dressing. Clotted or encrusted dressings can be moistened with Prontosan® Wound Spray in order to be easily removed and to avoid pain during removal. If the wound swells, becomes red or painful, or if the wound healing does not start in the course of a few days, you should consult your doctor.

**TRANSURETHRAL CATHETERS:** Thoroughly moisten a sterile gauze and cleanse the catheter entry site.

**PEG- / PEJ-TUBES AND SUPRAPUBICAL CATHETERS:** For cleansing of entry port, apply a sufficient amount of Prontosan® Wound Spray. Encrustations can be moistened with Prontosan® Wound Spray in order to be easily removed. Dry surrounding skin area with a sterile gauze and apply a sterile cover afterwards.

**STOMA CARE:** Thoroughly moisten a sufficient number of sterile compresses. Clean peristomal skin areas and dry afterwards. Prior to the use of Prontosan® Wound Spray, rinse off the residues of soaps, ointments and oils or other wound cleansing substances thoroughly.

**4. Intended users:** The Prontosan® Wound Spray is intended to be used by professional users or by adults. The intended patient population are adults and children.

**5. Side effects:** In very rare cases, there may be a mild burning sensation after application of Prontosan® Wound Spray, but this usually dissipates after a few minutes. Prontosan® Wound Spray can cause allergic reactions such as itching (urticaria) and rashes (exanthema). In very rare cases (less than 1 out of 10,000), anaphylactic shock has been reported.

**6. Prontosan® Wound Spray should not be used:** Contraindications:  
a) if the patient is known to be allergic or if it is suspected that the patient may be allergic to one of the ingredients of the product.  
b) on the central nerve system or the meninges.  
c) in the middle or inner ear.  
d) in the eyes. If Prontosan® Wound Spray comes into contact with the eyes, flush the eyes with running water and seek medical advice.  
e) on hyaline cartilage.

**7. Restrictions of use:** Pregnancy and lactation period: There is no evidence of mutagenic or embryo toxicity associated with the ingredients of this product. Due to the lack of relevant clinical trials and clinical experience with pregnant and breast-feeding women, Prontosan® Wound Spray should not be used.

**8. General safety instructions:** Do not use for infusion, injection or ingestion. Avoid inhalation of the Prontosan® Wound Spray aerosol. Do not use in combination with cleansing soaps (anionic surfactants), ointments, oils, etc. These substances should be thoroughly removed from the wound before use as they can affect the cleansing performance of Prontosan® Wound Spray. Do not use damaged bottles. Keep bottles out of direct sunlight. Keep out of reach of children.

**9. Summary / technical information:** Prontosan® Wound Spray is a preserved product and has a shelf life after opening 12 months. The spray head should be protected from contamination during use. Bottles that have come into direct contact with the wound or have become contaminated in another way must be discarded. The user and/or patient should report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

**Composition:** Purified Water, Betaine surfactant, 0.1 % Polymyripropyl Biguanide (Polihexanide)\*

**Appearance and smell:** clear, colour- and virtually odourless aqueous solution

**Shelf life:** according to the expiry date, store at room temperature \*if used separately, can be considered to be a medicinal product

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**B. Braun Australia Pty Ltd**
**B. Braun Medical AG**