Hansaplast

DE WUNDHEILSALBE

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Hansaplast

EN WOUND HEALING OINTMENT

Intended to support the healing of minor superficial wounds such as cuts, abrasions, scratches, 1st degree and smaller 2nd degree burns. Suitable for babies and damaged, irritated, dry or sensitive skin.

- Builds a breathable protective film that prevents the wound from drying out.
- Creates optimal, moist wound healing conditions, which promote the natural healing process of the skin and reduces the risk of scarring.
- Free of fragrances, colorants, preservatives and animal derived substances.

Hansaplast Wound Care Routine:

1. A clean and cleansed wound is the first step to an optimal healing. Cleanse your wound with the Hansaplast Wound Spray to protect effectively from wound infections.
2. The second step is the protection of the wound from external influences like dirt and bacteria to enable an undisturbed healing. Cover your wound with a plaster or sterile wound dressing from Hansaplast.
3. In a third step, you can help your wound heal safe, fast and with a lower risk of scars. Apply Hansaplast Wound Healing Ointment regularly until your wound has healed completely.

Directions for use:

- Stop the bleeding.
- Gently clean and dry the wound or skin area.
- Carefully apply a thin film one to two times per day. Make sure the opening of the tube does not get in contact with your wound.
- Repeat regularly until the healing is completed.
- Can be used in any stage of the healing process.
- If necessary, cover the wound with a plaster, dressing or compress. Change daily if not recommended otherwise by your doctor.

Warnings and precautions:

- In case of larger wounds with heavy exudate or signs of infection a doctor should be consulted.
- In case of burns, contact a doctor in case of doubts to determine the severity of the burn.
- For external use only.
- If redness, swelling or pain develops at the wound area, the condition gets worse or the wound healing process does not start within several days a doctor should be consulted.
- In case of pregnant or breastfeeding women, consult a doctor before use.
- Do not use in combination with condoms out of latex as Hansaplast Wound Healing Ointment can reduce the tear strength and therefore the safety of condoms.

Contraindications:

Do not use:

- on mucous membranes and in or near the eyes.
- in combination with other topical preparations in the same area.
- in combination with anionic surfactants, soap, oils, gels, etc.
- in case of known allergies to any of the ingredients.
- in case of an allergic reaction stop using Hansaplast Wound Healing Ointment.
- continuously for more than 30 days.
- Do not swallow.

Side effects:

It cannot be excluded that in some rare cases a skin irritation or a rash occurs during usage. If you observe such a reaction or in case of other symptoms discontinue use and consult a physician.

If you observe unexpected or potentially serious symptoms following the usage of this product, please inform your doctor or pharmacist, the manufacturer and your national health authority.

Composition:

White Petrolatum, Thin Paraffin Oil, Ceresin Wax, Glycerin, Panthenol, Glyceryl Stearate

Packaging sizes:

Aluminum tubes in 3.5g, 20g, 50g, 100g. Possibly not all packaging sizes are registered for sale in all countries.

Storage information:

Keep the instructions as long as you are using the product. Store between 5°C - 30°C in the original packaging. Close tube after usage. Only use undamaged tubes. Expiry date and lot number are engraved in the tube fold. Only use until expiry date. Use within 6 months after opening. Keep out of the reach of children.

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